## INSTRUCTIONS FOR USE, CLEANING, AND STORAGE FOR REUSABLE PVC MANOMETRIC WATER-PERFUSED AND/OR BAROSTAT CATHETERS

### INSTRUCTIONS FOR USE

- Post-procedure manometric catheters are contaminated with body fluid.
- Only use catheters that have been cleaned and high-level disinfected or sterilized according to instructions below.
- Proper aseptic technique and universal barrier precautions (UBP) must apply.
- Do not use** silicone-based lubricants on the catheter if it has a balloon.
- Inspect catheter and packaging for any mechanical damage or imperfections.  
**Do not use if damaged.**
- This catheter must only be used under medical supervision by personnel who have been adequately trained in safe gastro-intestinal intubation, and in the performance of perfused manometric measurements and/or barostat measurements.

### CHEMICAL CLEANING / DISINFECTING INSTRUCTIONS

#### 1. Cleaning

- If catheter has a balloon attached, plug the end of the luer connector to ensure no water (or other liquids) enters into the balloon.
  - Wash catheter in warm water with enzymatic detergent solution.
  - Flush detergent through all channels including unused channels. Remove all foreign matter. Wipe and clean external surface.
  - Rinse catheter and flush all channels with clean fresh water to remove detergent. Repeat as needed.
  - Blow-dry all channels with air and wipe excessive water from external surface.
- #### 2. Chemical Disinfection (high-level disinfection)
- Flush disinfectant through all channels including unused channels. Ensure the entire catheter is submerged in disinfectant and all channels are filled with disinfectant.
  - Soak catheter in disinfectant solution according to solution manufacturer's instructions.

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### ETO GAS REPROCESSING

- Reprocessing by Ethylene Oxide gas has been validated effective on these manometric catheters.
- Ensure pigtail luer ends are open (remove all luer caps) prior to reprocessing.
- Reprocess at a temperature of 55°C for a minimum full-cycle of 180 minutes.
- Aerate at a temperature of 51-59°C for a minimum of 12 hours.

### STORAGE

- Ensure all channels and external surfaces are completely dry. Store catheter dry in open air. **Do not store in a sealed container.**
- Storage conditions:  
Temperature: 15°C - 30°C (60.8°F - 86.6°F)  
Relative Humidity: 10% - 75%

**Lifetime of product – 5 years or 50 uses whichever occurs first.**

**Manufacturer's Warranty – 6 months from Manufactured Date for any manufacturer's defects.**

#### Caution:

Do **not** soak catheter in alcohol.  
Do **not** steam autoclave catheters.

*Mui Scientific is not liable for any damage to the catheter, or harm to patients or personnel, caused by improper use of a disinfectant or procedure.*

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Authorized Representative in EU: Advena Ltd.