INSTRUCTIONS FOR USE, CLEANING, AND STORAGE FOR MANOMETRIC WATER-PERFUSED AND/OR BAROSTAT CATHETERS

INSTRUCTIONS FOR USE

- Post-procedure manometric catheters are contaminated with body fluid.
- Only use catheters that have been cleaned and high-level disinfected or sterilized according to instructions below.
- Proper aseptic technique and universal barrier precautions (UBP) must apply.
- **Do not use** silicone-based lubricants on the catheter if it has a balloon.
- Inspect catheter for any evidence of mechanical damage or imperfections.
Do not use if damaged.
- This catheter must only be used under medical supervision by personnel who have been adequately trained in safe gastro-intestinal intubation, and in the performance of perfused manometric measurements and/or barostat measurements.

CHEMICAL CLEANING / DISINFECTING

INSTRUCTIONS

1. Cleaning

- If catheter has a balloon attached, plug the end of the luer connector to ensure no water (or other liquids) enters into the balloon.
- Wash catheter in warm water with enzymatic detergent solution.
- Flush detergent through all channels including unused channels. Remove all foreign matter. Wipe and clean external surface.
- Rinse catheter and flush all channels with clean fresh water to remove detergent. Repeat as needed.
- Blow-dry all channels with air and wipe excessive water from external surface.

2. Chemical Disinfection (high-level disinfection)

- Flush disinfectant through all channels including unused channels. Ensure the entire catheter is submerged in disinfectant and all channels are filled with disinfectant.
- Soak catheter in disinfectant solution according to solution manufacturer's instructions

Caution:

- The catheter contains Polyvinyl Chloride (PVC), Polycarbonate, Stainless Steel (if a balloon is attached it also contains Polyisoprene or Polyolefin, and Silk sutures). **Check with disinfectant manufacturer to ensure their disinfectant can be used safely on the materials listed above.**
- Cidex-OPA (Johnson & Johnson), Sporox (Sultan Chemists Inc.), Korsolox Extra (Bode Chemie GmbH & Co.), OPTIM CS (SciCan Ltd.) and Adaspor Plus Ready to Use (IMS Group) are compatible with the catheter and should be used according to the solution's manufacturer's instructions.
- **DO NOT USE CIDEX PRODUCTS TO DISINFECT ANY MUI SCIENTIFIC CATHETER THAT HAS A BALLOON.**

3. Final Rinsing and Blow Drying

- Rinse catheter and flush each channel with sterile water to remove disinfectant. Repeat as needed.
- Ensure all disinfectant is completely removed.
- Blow-dry all channels with air and wipe all water from external surface.

ETO GAS STERILIZATION

- Sterilization by Ethylene Oxide gas has been validated effective on these manometric catheters.
- Ensure pigtail luer ends are open (remove all luer caps) prior to sterilization.
- Sterilize at a temperature of 55°C for a minimum full-cycle of 180 minutes.
- Aerate at a temperature of 51-59°C for a minimum of 12 hours.

STORAGE

- Ensure all channels and external surfaces are completely dry.
- Store catheter dry in open air. **Do not store in a sealed container.**

Shelf-life: 2 years for catheters with balloons and 3 years for catheters without balloons.

Manufacturer's Warranty – 6 months (or 50 uses whichever occurs first) from Manufactured Date for any manufacturer's defects.

Caution:

- Do not soak catheter in alcohol.
- Do not steam autoclave catheters.

Mui Scientific is not liable for any damage to the catheter, or harm to patients or personnel, caused by improper use of a disinfectant or procedure.

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